COAMATIC® Protein C - 82 2098 63

**Intended use**

This kit is for the quantitative determination of Protein C activity in human citrated plasma.

Background and summary

Protein C is a vitamin K dependent plasma protein which plays an important role in the anticoagulant regulatory mechanisms. It circulates as a zymogen and is converted to an active serine protease, activated Protein C (APC), by the action of thrombin in presence of thrombomodulin. APC regulates the coagulation system by proteolytic cleavage and inactivation of activated factors V and VIII. Hereditary and/or acquired Protein C deficiency has been shown to be a risk factor for development of venous thrombosis.

Measurement principle

Protein C in plasma is activated by a specific enzyme from Southern Copperhead Snake venom. The amount of activated protein C is determined by the rate of hydrolysis of the chromogenic substrate S-2366. The pNA release measured at 405 nm is proportional to the Protein C level in the range from 0-120% of normal plasma.

**Reagents**

1. S-2366 6 mg 2 vials
Lyophilized chromogenic substrate pyroGlu-Pro-Arg-pNA-HCl.

2. Protein C activator 1.2 U 2 vials
Lyophilized venom enzyme from Southern Copperhead Snake (Agkistrodon Contortrix Contortrix), with bovine serum albumin (stabilizer) and Ciprofloxacin® (preservative).

PRECAUTION AND WARNINGS

S-2366:

- Hazard class: **none**
 - Risk phrases: **none**
 - Safety phrases: **none**
- Protein C Activator:
- Hazard class: **Xn** (Harmful)
 - Risk phrases: **May cause sensitization by inhalation and skin contact (R42/43)**
 - Safety phrases: **Do not breathe dust (S22)**
Wear suitable protective clothing and gloves (S36/37)
In case of accident or if you feel unwell, seek medical advice immediately (show the label where possible) (S45)
In case of accident by inhalation: remove casualty to fresh air and keep at rest (S63)

Protein C activator 1.2 U contains bovine material. All donor animals were sourced from BSE-free herds. The cattle received ante- and post mortem health inspection by a veterinarian, and they were apparently free from infectious and contagious material. However, the material should be treated as potentially infectious.
This product is for *in vitro* diagnostic use.

Reagent preparation

For microplate and test tube techniques reconstitute each reagent with 7.2 mL of water (see REAGENTS 3.) Replace the stopper and swirl gently. Make sure of the complete reconstitution of the product. Keep reagent at 15-25°C for 10-30 min and invert before use.

NOTE: Other reagent reconstitution volumes may apply for automated methods. Reagents are not interchangeable between lots.

Reagents required but not provided

3. Deionized water, filtered through 0.22 µm or NCCLS type II water^a.

4. Human normal plasma with a Protein C activity of 100%.

Different standard dilutions are obtained by diluting normal plasma with water (see REAGENTS 3). A lyophilized preparation of human normal plasma, calibrated against International standard for Protein C is recommended.

5. Acetic acid 20% or citric acid 2%.

Materials required but not provided:

— Spectrophotometer 405 nm (405 nm and 490 nm for microplate procedure)

— Incubator 37°C ± 0.2°C

— Microplates[®] or semi-micro cuvettes

— Centrifuge, 2000 x g

— Plastic test tubes

— Stopwatch

— Vortex mixer

— Calibrated pipettes


— Linear graph paper

*NOTE: Do not use microplates intended for coating!

ENGLISH - Insert revision 08/2011

Calculation

Plot the absorbance (A) for the standard samples against their protein C activity on linear graph paper. Plot A on the Y axis and % protein C on the X axis. Connect the standard points with the best fitting straight line. Samples are evaluated based on this standard curve. An example of a typical standard curve (microplate method) is shown below.

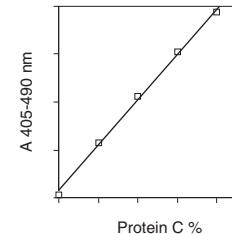Protein C %

Performance Characteristics**Limitations/interfering factors**

Addition of anti-human Protein C antibodies to normal plasma inhibits >98% of the amidolytic activity. Heparin levels ≤ 3 IU/mL do not interfere in the assay. Aprotinin inhibits activated Protein C. A low Protein C activity in aprotinin treated patients is thus expected.

Sample blank activities should be determined and subtracted in plasmas from patients on Streptokinase therapy as well as in plasmas where contact factor activation is suspected, such as in plasmas from DIC patients or from individuals on oral contraceptives where cold activation may occur. The sample blank activity is determined by substituting the Protein C activator with the same volume of sterile water.

Precision

The repeatability of the method has been determined on Cobas Mira.

Mean activity	Within run CV (%)	Mean activity	Between runs CV (%)
99%	0.8% n=21	103%	1.7% n=5 N=7
47%	1.6% n=21	49%	2.8% n=5 N=7

n=number of replicates per series, N=number of series

These results have been obtained at Chromogenix laboratories and should be considered as examples only. Other laboratories are requested to establish their own precision data.

Correlation

The assays show a strong correlation with Coatest Protein C:

% PC (Coamatic) = 8.2 + 0.99x (Coatest) r=0.98 n=45

% PC (Cobas Mira) = 3.7 + 0.98x (Coatest) r=0.99 n=49

% PC (Cobat Mira) = 3.7 + 0.98x (Coatest) r=0.97 n=45

% PC (ACL) = 0.3 + 1.07x (Coatest) r= 0.97 n=45

Recommended measuring range

The relationship between released pNA, measured as absorbance at 405 nm, and the level of Protein C is linear in the 0-120% range of normal plasma.

Detection limit

The assays allow detection of at least 5% Protein C.

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- STOCKNER K et al. Protein C activators in snake venoms. Behring Inst Mitt 79, 37-47 (1986).
- EXNER T and VAAJOKI R. Characterisation and some properties of the Protein C activator from Agkistrodon Contortrix venom. Thromb Haemost 59, 40-44 (1988).
- PABINGER I. Clinical relevance of Protein C. Blut 53, 63-75 (1986).
- VINAZZER H and PANGRAZ U. Protein C: Comparison of different assays in normal and abnormal plasma samples. Thromb Res 46, 1-8 (1987).
- WENDEL H P et al. Apronin in therapeutic doses inhibits chromogenic peptide substrate assays for Protein C. Thromb Res 74, 543-548 (1994).
- National Committee for Clinical Laboratory Standards. Specifications for reagent water used in the clinical laboratory. NCCLS Approved Standard: ASC-3.
- National Committee for Clinical Laboratory Standards. Collection, transport and processing of blood specimens for coagulation testing and performance of coagulation assays, NCCLS Document H21-A2; vol. 11 No. 23.
- ZUCKER S, CATHEY M H, WEST B. Preparation of Quality control specimens for coagulation. Am J Clin Pathol 53, 924-927 (1970).
- WESTGARD J O, BARRY P L. Cost-effective quality control: Managing the quality and productivity of analytical process. AACC press (1988).

Symbols used / Verwendete Symbole / Símbolos utilizados / Symboles utilisés / Simboli impiegati / Símbolos utilizados / Χρηματοποιηθεντά σύμβολα

IVD

In Vitro Diagnostic Medical Device

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Instrumentation Laboratory SpA - V.le Monza 338 - 20128 Milano (Italy)