



## ELISA Protein C

---

Capture and detecting antibodies for the construction of an ELISA for the quantitative determination of Protein C.

### Test principle

Micro wells are coated with Protein C antibodies (capture antibody), to which Protein C from the sample binds. Unbound substances are removed by washing. A peroxidase conjugated detecting antibody is added and binds to the Protein C/capture antibody complex. After a washing step, added enzyme substrate reacts with the conjugate. The intensity of the resulting colour is proportional to the Protein C content in the sample.

### Reagents and stability (after opening)

Capture antibody 1 vial

Detecting antibody 1 vial

When stored at -20°C the reagents remain stable until the expiry date.

**No. of determinations:** 5 x 96

**Article no.:** PC-EIA